

Demonstrated Clinical Performance

EASY-R Clinical Study (n=39)¹:

- 100% successful epicardial access
- Mean time to access: 3 ± 3 minutes
- 20 of the 39 procedures (51.3%) achieved access in 2 minutes or less
- Low fluoroscopy time (2.0 ± 1.3 min) and minimal contrast use (0.7 ± 0.5 mL)

¹"Epicardial Access Study with Rook (EASY-R) - Final Clinical Study Report" - CIRCA internal test report. ClinicalTrials.gov ID: NCT06388629

Ordering Information

PeriCross™ Epicardial Access Kit Product Code: PC-1013

Indications for Use

The PeriCross™ Epicardial Access Kit is intended to access the epicardial surface of the heart via a subxiphoid approach to facilitate electrophysiology studies in adult patients.

Rx Only Caution

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Patents: www.circascientific.com/en-us/patents

© 2025 CIRCA Scientific, Inc. All Rights Reserved. PeriCross is a trademark of CIRCA Scientific, Inc. The third-party trademarks used herein are trademarks of their respective owners.

PC-ART100 Rev. 00



CIRCA
SCIENTIFIC

CIRCA Scientific, Inc

Corporate Office 14 Inverness Drive East, Suite
H-136, Englewood, CO 80112

info@circascientific.com

Office: 1.303.951.8767

Fax: 1.303.951.8769

www.circascientific.com



PeriCross™ Epicardial Access Kit

Built for procedural confidence and control




The Challenge of Epicardial Access

Epicardial access remains one of the most technically demanding steps in electrophysiology procedures. The heart's constant motion, delicate anatomy, and lack of visual or tactile feedback contribute to a challenging and stressful environment.

Key Considerations:

- Thin, mobile pericardium (~1 mm)
- Risk of cardiac and extra-cardiac complications, including RV puncture, vascular injury, or bleeding
- Conventional tools lack precision and predictability

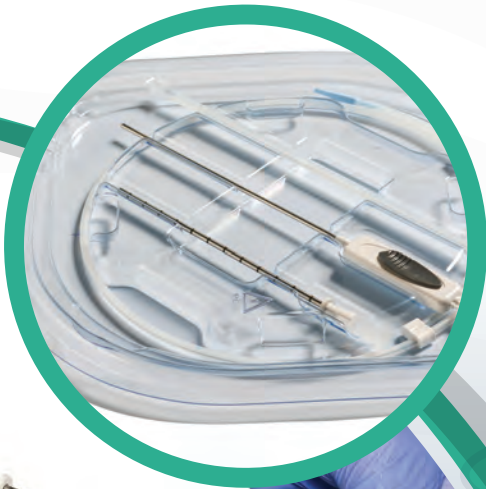

The PeriCross™ Solution

The PeriCross™ Epicardial Access Kit is purpose-built to support efficient, controlled access to the epicardial space via a subxiphoid approach. It combines intuitive design with integrated safeguards to assist physicians during this critical procedure-helping streamline workflow and support timely access.

System Features:

- Tine-based pericardial capture to create separation before puncture
- 21G micro-puncture needle designed to minimize trauma
- Interlock prevents needle deployment until tines are engaged
- Coaxial introducer for seamless guidewire exchange

Kit Includes:

- PeriCross Tunneler Assembly
- PeriCross Access Device
- .018" x 80cm Guidewire
- 5F x 18cm Coaxial Introducer



OUTWARD RETRACTION
OF PERICARDIUM